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4

Place a new inspection label, stating next inspection in month and year, on the MARK Infini Box.

DANGER

If the lead seals are inadvertently getting removed, you have to carry out a direct check of the MARK Infini Box (max. 6hours later). In this special case, there is no need of verification by the manufacturer. The incident must be documented.

TIP

TIP

Specific storage conditions make regular opening of the MARK Infini Box obsolete. If the environmental influences allow it (dry, protected from mechanical and chemical influences, etc.), only the existing of the seals and a visual inspection need to be carried out and documented during the annual inspection! The competent person has the responsibility of the assessment!

In doubt of the safe functions of a component in the MARK Infini Box you can contact the manufacturer or you can send back the box to:

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3.1 Inspection steps MARK Infini Box

1 Check if there is no visible damage outside on the MARK Infini Box.

2 By tearing off the strap you can remove the lead seal and open the MARK Infini Box.

3 Check if the removed lead seals are ident with the serial numbers on the test protocol.
If the serial numbers do not match please contact the manufacturer.

4 Check the remaining lead seals in the lead seal pack (the numbers of the lead seals must be twice the amount of the remaining years). If some are missing or do not match, please contact the manufacturer.

5 Check if there is no visible damage on the MARK Save A Life products or on other PPE products inside the box. (Check if the products are still protected from moisture by condensation, humidity and UV-radiation).

1 Take two new lead seals and document the new serial number on the test protocol inside the box.

2 Register the inspection in the test protocol, with date of inspection, result and signature.

3 Seal the MARK Infini Box with the two new lead seals as shown on the picture.

2 PRODUCT OVERVIEW

Symbol photo

2.1 Scope of delivery

- MARK Infi! Box
- Personal Protective Equipment (PPE) acc. your order
- 2 Lead seals outside
- 1 Rotary Pack with 18 lead seals (when buying a new one)
- 1 Testprotocol
- 1 Desiccation bag
- 1 Instruction for use
- 1 Inspection Label

2.2 Marking

The buckles of the MARK Infi! Box are sealed with lead seals.

Manufacturer's logo
Seriennummer

- Use an inventory number for this product. Enter the inventory number in the protocol.
- Do not remove labels or other markings from this product.

TIP

The rescue devices and accessories by MARK Save A life are certified personal protective equipment (PPE) in accordance with PPE regulation 2016/425 and are subject to periodic inspection rules.

Therefore the MARK Infi! Box must be inspected and checked at least once every 12 months and documented in accordance with the test protocol inside the box. The inspection must be carried out by the manufacturer or a competent person .

After ten years the MARK Infi! Box must be taken out of service and have to be checked from the manufacturer.
We can therefore extend the lifetime of the MARK Infi! Box for another 10 years.

3 INSPECTION

1 FOR YOUR OWN SAFETY

It is possible to use this product incorrectly in many ways and we cannot possibly list or think of all of them.

These instructions for use will give you information on your safety and the product and its use. This information and the safety instructions do not however replace personal experience, knowledge of the product and self-responsibility.

You will find detailed safety regulations and descriptions in the following chapter and at the beginning of each chapter. Read them carefully and make sure you understand them. If you need more information or if you are not sure how to use this product, get in touch with MARK Save A Life. We will be happy to help you.

This product is state of the art and corresponds to all applicable technical standards and guidelines.

1.1 Symbols used

In this document safety regulations of the following categories are being used: **DANGER, WARNING, CAUTION** and **TIP**. Follow the safety instructions to avoid personal injuries or damage of the product.

DANGER means that death or a serious injury are very likely to happen if an accident occurs or a hazardous situation is not avoided.
 • In order to avoid serious injury or death all safety instructions marked with **DANGER** must be followed.

WARNING means that death or a serious injury are likely to happen if an accident occurs or a hazardous situation is not avoided.
 • In order to avoid serious injury or death all safety instructions marked with **WARNING** must be followed.

CAUTION means that moderate or minor injuries may happen if an accident occurs or a hazardous situation is not avoided.
 • In order to avoid moderate or minor injuries all safety instructions marked with **CAUTION** must be followed.

TIP draws your attention to a possible damage of the product or other properties if an accident occurs or a hazardous situation is not avoided.
 • In order to avoid damage of the product or other properties all safety instructions marked with **TIP** must be followed.

DANGER

WARNING

CAUTION

TIP

- 1.2 Storage**
- Store this product in a dry and clean place protected from mechanical (e.g. sharp edges) and chemical hazards (e.g. acids, gas) at room temperature and protected from light (e.g. UV).

1.3 Maintenance

- Repairs may only be effected by MARK Save A Life or an authorized partner.
- This product (the rope in particular) must not be disinfected or impregnated.
- Do not use chemicals when cleaning the product.

NOTE

In order to make our texts easier to read, we dispense with gender differentiation and write main personal words in the male form. This is meant as a neutral wording by which we address all people equally without any discrimination.
The shown pictures are examples and symbol photos.
The actual contents of the MARK Infini Box may differ due to other compilations.

MARK Infini Box

MARK Infini Bag

TERMINOLOGY

COMPETENT PERSON

trained according to DGVV 312-906 or with appropriate regional training

AUTHORIZED PARTNER

trained by MARK Save A Life. The authorized partner is authorized to perform repairs on this product.

1 FOR YOUR OWN SAFETY

1.1 Symbols used

1.2 Storage

1.3 Maintenance

2 PRODUKTBESCHREIBUNG

2.1 Scope of delivery

2.2 Marking

3 INSPECTION

3.1 Inspection steps MARK Infini Box

3.2 Re-Sealing the MARK Infini Box

5
5
6
6
7
7
8
9
10

Congratulations on purchasing this product. It was manufactured and tested subject to strict quality standards and with the utmost care. We thus laid the groundwork for safe use. It is now your responsibility to use this product correctly.

For correct and safe use of this product it is essential that you read these instructions carefully. Follow all safety instructions in the respective chapters of this manual.

You may only use this product if you have read and understood these instructions for use.

This manual including the inspection protocol must be kept with the product. In case of loss or if manual and inspection protocol became unusable, the effective versions of both documents are available for download at www.saveallife.at

You can download the instructions for use and the test protocol via www.saveallife.at.

Do you have any questions or wishes concerning this product or do you want to give us feedback? Just call us or send an email.

Dear customer,

TIP

Customer service

MARK Save A Life GmbH
Gleinkerau 23
4582 Spital am Pyhrn
Austria / Europe
Tel +43 7563 8002 – 111
Fax +43 7563 8041 – 111
mark@saveallife.at
www.saveallife.at

Manufacturer

MARK Save A Life GmbH, Gleinkerau 23, 4582 Spital am Pyhrn, Austria / Europe

Country of origin

Austria / Europe

Copyright

MARK Save A Life GmbH, Gleinkerau 23, 4582 Spital am Pyhrn, Austria / Europe

MARK Infi! Box Instruction for Inspection

The compact packaging provides more protection of our rescue devices and other PPE

MARK Save A Life GmbH
Gleinkerau 23
4582 Spital am Pyhrn
Österreich / Europa
Tel +43 7563 8002 - 111
Fax +43 7563 8041 - 111
mark@savealife.at
www.savealife.at

Translation / German into English
Keep for future use
Version 1.1